CarboEye Tea

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 ITALDEVICE Srl - Via Laurentina Km 26.700 - 00071 Pomezia (RM) Italia

CarboEye Tea

Lubricating Ophthalmic Solution with Hypromellose

Dry eye, a widespread phenomenon, is often responsible for damage to the surface of the eye. It is manifested with irritation, itching, burning, and tiredness, and may be linked to variety of causes—either environmental, such as excessive exposure to sunlight, prolonged use of video terminal, and air-conditioned environments, or as a consequence of eye pathologies. The use of Hypromellose-based ophthalmic solution specially formulated to treat dryness and for corneal protection, ensures, with its high viscoelasticity, the right degree of hydration, thus alleviating the symptoms associated with eye dryness. The presence of distilled waters of plant origin improves corneal hydration.

Indications:

Lubricating ophthalmic solution indicated for the symptomatic treatment of dry eye. Also indicated for corneal protection as tear substitute.

Method of use:

Administer one or two drops in each eye one or more times a day or as recommended by your physician.

Compositions:

Green tea water, chamomile water, hypromellose, EDTA disodium salt, potassium sorbate, buffered isotonic solution pH 7,2.

Warning:

Do not use the product if you are allergic to or have a known hypersensitivity to any of its ingredients

- The product is sterile until the bottle is opened.
- Close the bottle tightly after use.
- Do not use the product 60 days after it is opened.
- Do not use the product after the expiration date indicated on the bottle.
- Keep away from heat sources and out of reach of children.
- Do not ingest.
- Do not use the product if the packaging has been compromised.
- In the event of an adverse reaction, suspend use of the product and consult your physician.
- Take care that the tip of the bottle does not come into contact with the eye.
- Do not dispose of the bottle into the environment after use.

CE 1370

STERILE A

10 ml  

Distributore: 360 Oftal SRL, Via Lombardia 6/b, 90144 Palermo, IT
 ITALDEVICE Srl - Via Laurentina Km 26.700 - 00071 Pomezia (RM) Italia